

## 6.0 SmartMan Manual – 6. Actions Menu 0. Overview

---

Current version v2.0

Click on any row in **Table of Contents** to go to that section

---

### Table of Contents

6.0 SmartMan Manual – 6. Actions Menu 0. Overview .....	1
6.0 The Actions Menu .....	1
6.1 Register .....	2
6.2 Login Next User .....	2
6.3 View Previous Results .....	2
6.4 Check for Updates .....	2
6.5 Calibrate SmartMan .....	2
6.6 Std. Quick Start ON .....	2
6.7 Adv. Quick Start OFF .....	2
6.8 SmartMan ONLINE .....	2
6.0.1 Active and Inactive Menu Options.....	2

---

### 6.0 The Actions Menu

The Actions Menu is where you perform actions which are required by the program. Here a new login registers and can view previous results of activities and tests performed. It also contains buttons which can speed up how you get through the Skills Menu.



## 6.1 Register

Every person who logs in must be registered before the skills in the Actions and Trainer Menu are available. When a person has a valid registration, this will be grayed out.

## 6.2 Login Next User

This returns you to the login menu so that another person can log in to the program.

## 6.3 View Previous Results

This allows the person logged in to view previous results of activities and tests that they have completed. This menu also allows you to print results at a later time.

## 6.4 Check for Updates

You must be connected to the internet to use this feature. It will check to see if you have the latest version of the software or not.

## 6.5 Calibrate SmartMan

SmartMan now calibrates each time you go to the Main Menu. Click this button to see the current sensor readings and to force it to calibrate.

## 6.6 Std. Quick Start ON

When ON, this button reduces the number of options you click before you get to a SmartMan activity. Once you know which activity you wish to use this can save time. Click on this button and it will turn off so that you have the options to select how you wish to do an activity and a test.

## 6.7 Adv. Quick Start OFF

This button only applies to Advanced Airway Activities. When this button is OFF, when you select an Advanced Airway Activity or Test you have all of the options to select. When ON this button reduces the number of options you click before you get to a SmartMan activity. Once you know which activity you wish to use this can save time. Click on this button and it will turn off so that

## 6.8 SmartMan ONLINE

If SmartMan is online, the program will attempt to send results to the server as soon as an activity is completed. If there is no internet connection it will not send results. If you know ahead of time that you do not have an internet connection, click this button to turn it OFFLINE. This will save time.

[Top](#)

---

### 6.0.1 Active and Inactive Menu Options

Selections in the Actions Menu are only available when they are showing in dark blue font. Reasons they will be grayed out (lighter blue font) include:

The person logged in has NOT registered. When this person registers the option will become available.

The person logged in has registered but has exceeded the number of practices or tests allowed. Re-register or log in a different user and these will become available.

Calibration readings are outside of what is acceptable.

[Top](#)